

LINFOPRESS
EVOLUTION PRO



The results obtained after the study carried out show the effectiveness of the treatment with LINFOPRESS EVOLUTION PRO for

LYMPHATIC AND CIRCULATORY DRAINAGE

SLIMMING
TREATMENTS

BODY CONTOURING

POST-LIPOSUCTION

ADVANCED
COMPRESSION
THERAPY

PRE-LIPOSUCTION

HEAVY LEGS

ANTI-CELLULITE

TERMO SALUD

TECHNICAL DETAILS

LINFOPRESS
EVOLUTION PRO
Ref. M86




Power supply:	230V a.c. 50 / 60 Hz 115V a.c. 50 / 60 Hz (optional)
Maximum consumption:	180VA (0,75A to 230Vac)
Output characteristics:	Maximum output pressure: 80mmHg
N° chambers/garment:	jacket: 24 trousers: 24
Pressure time:	0,5 - 25 sec
Treatment time:	20/30/45 min. aprox.
Classification:	Class I type BF
Protection fuses:	2 x T1,5A / 250 VAC
Protection degree:	IPX0
Interface:	Colour Touch screen 10.2" Blacklight LED

Dimensions:	50 cm (h) x 44 cm (l) x 50 cm (w)
Weight:	18 Kg

TERMO SALUD

ACCESSORIES INCLUDED

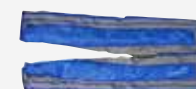

1 x
PRESSOTHERAPY
TROUSERS
REF: A0151



1 x
PRESSOTHERAPY
JACKET (optional)
REF: A0150



1 x
POWER
CABLE
REF: SUB-221



1 x
REMOTE
CONTROL
REF: A430

grupo BSB



© 2016 Termosalud. Marketing Dpt.

TERMO SALUD
MEDICAL
& ESTHETIC

P. Ind. Los Campones. C/ Ataúlfo Frieria Tarfe nº 8. Tremeñes 33211 Gijón. ASTURIAS
Tel. 902 10 22 25 info@termosalud.com www.termosalud.com



PRESSOTHERAPY

advanced compression therapy



TERMO SALUD
MEDICAL
& ESTHETIC



HOW DOES LINFOPRESS EVOLUTION PRO WORK?

LINFOPRESS EVOLUTION PRO is the result of the more than 30 years' experience of Termosalud in developing highly effective pressotherapy devices for the treatment of lymphatic and circulation problems.

It acts on the lower part of the torso, lower limbs and abdomen.

And thanks to the incorporation of a pneumatic jacket, it also works on the upper lumbar, thorax, back and abdominal region as well as the upper limbs.



INDICATIONS

LINFOPRESS EVOLUTION PRO helps to treat various medical-beauty disorders by stimulating the blood and lymphatic circulation.

It is the ideal complement for beauty treatments aimed at very specific problems like cellulite, obesity disorders, circulation and varicose problems, or water retention and oedemas.

It is also effective in post-operative treatments and for surgical preparation and recovery, post-natal recovery, muscle discomfort, painful joints and posture correction; as well as for states of stress, insomnia and anxiety.

BENEFITS

- Stimulates the circulation and lymphatic systems, simply, quickly and effectively
- Improves the circulation and alleviates the feeling of tired, heavy legs.
- Helps to eliminate toxins naturally, and is therefore the ideal complement for inch-loss and cellulite treatments.
- Reduces bloating and helps to eliminate toxins
- Ideal for body contouring
- Helps to increase blood flow and, consequently, improves the tone and firmness of skin
- Painless and non-invasive

OBJECTIVE OF THE PRESSOTHERAPY

The objective of **LINFOPRESS EVOLUTION PRO** is to guarantee therapeutic well-being for the body, stimulating the circulation and lymphatic system in each specific zone of the body and eliminating an excess of superfluous interstitial fluids, relaxing the body and achieving an increased state of well-being.

MODE OF ACTION OF THE PRESSOTHERAPY

The system consists of a set of pneumatic garments that adapt to the area and volume of the zone to be treated. Additionally, thanks to the intermittent compression method, the pressure is adjusted to suit each area of the body.

Our equipment applies alternate compression to different points of the body, to soothe and correct any disorders caused by poor functioning of the lymphatic and circulation systems.

LINFOPRESS PRO. OVERLAPPING CELLS SYSTEM

The pressure cells of the **LINFOPRESS EVOLUTION PRO** are distributed to overlap each other which enables all the zones of the body to be compressed to ensure more comprehensive drainage. The slanted layout of the cells means the lymph is sent horizontally and upwards, pushing it and guiding it towards the lymph node points of the body, preventing it from spreading to other areas.

The design of the **LINFOPRESS EVOLUTION PRO** trouser garment adapts completely to the treatment zone, from the feet right up to the abdomen, to apply pressure evenly over the whole surface. The trouser garment is fitted with 24 air chambers, which are spread out evenly over the whole treatment zone. It is fastened using zips that are adjustable to fit the patient's body and figure, and extenders can be fitted to make the garment bigger if necessary.

LINFOPRESS EVOLUTION PRO has a specific jacket for treating the arms, back, thorax and abdomen that boosts the effectiveness of the pressotherapy in the overall treatment. The device is suitable for use in the medical sector; it achieves the patient's expectations as well as those of the beautician-medical professional, as the treatment can be paused and restarted, and some cells can be left unused during treatment.



LINFOPRESS PRO FUNCTIONING CYCLES

LINFOPRESS EVOLUTION PRO has 4 preset cycles:

• Pre-Treatment.

Designed to open the lymphatic channels. This should always precede the other cycles and it is very useful for treating cellulite or lymphatic oedemas.

• Wave

It applies a compression sequence that moves the lymph in the zone to be treated (peristaltic compression), in a distal to proximal direction gradually in all the chambers. The massage effect this achieves is suitable for treating cellulite, applying a sports massage, pre-liposuction and muscle relaxation.

• Relaxing

It works on each chamber for the same amount of time. This is the ideal way to finish treatments for cellulite or oedemas, tired legs, post-liposuction, pregnancy and post-natal, menopause, etc.

• Lymphatic

It applies complete waves in a distal to proximal direction, with the pressure decreasing as the wave reaches the proximal zone (Pecquet's cistern and the heart). The pressure in the chambers can be adjusted individually. This is ideal for treating lymphatic oedemas.



LINFOPRESS EVOLUTION PRO FUNCTIONING MODES

PROGRAMME MODE

The combination of these four functioning cycles provides a wide variety of medical-beauty treatments, each of which is suitable for treating a specific problem: cellulite, water retention, flaccidity, circulation, lymph, post-natal, pre and post liposuction, among others.

It is very simple to select the programmes using the practical and easy to manage operating system, which indicates, in each of the sections, the steps to follow to guarantee correct programming. Each of them can be set for the different phases comprising the pressure, speed, length of time taken to pass from one chamber to the next, and the resting time between cycles. In the Lymphatic phase, the pressure setting can be adjusted separately for each chamber.

FREE MODE

Based on the combination of the four preset cycles, it is possible to create up to ten personalised treatments, to suit the specific needs of each client. Additionally, these treatments can be stored in the equipment memory, and they can be given a number to identify them. As with personalised programmes, all the treatment settings can be adjusted for each cycle included in the programme.