



DEC 30 UP

ENDOSHAPER

device for endodermic massage

Decomedical di J.M.L. s.r.l.

Via Lauro, 5 - 84090 - S. Mango Piemonte - Italy

Tel. + 39 089 631094 | Fax + 39 089 631333

www.decomedical.com



Endoshaper is a device for endodermic massage performed by means of compression and distension movements. The mechanical endodermic massage consists in an elective action of the subcutaneous connective tissue by using a vacuum pressure system associated with the pressure of rollers mounted inside the handpiece. The vacuum pressure exerted by the handpiece can be continuous and/or intermittent. Thanks to the handpieces and applicators, the skin and subcutaneous tissue are lifted and massaged in depth.

Intended use:

- Compact cellulite
- Flaccid cellulite
- Oedematous cellulite
- Lymphatic drainage
- Localized adiposity
- Asphyxiated skin
- Hypotonicity

ACCESSORIES



AC0662

Endoshaper handle



AC0695

vacuum handle



AC0695-C

large vacuum cup



AC0695-B

medium vacuum cup



AC0695-A

small vacuum cup



AC0695-D

5 handpiece filters



AC0710

Tablet



AC0697

rubber tube for
Endoshaper handle

The Endoshaper device used for body treatments allows to perform, in a single treatment, three specific actions such as: lipolytic action, toning action and draining action, with excellent results visible from the first session.

Lipolytic action: due to the suction effect of the vacuum pressure, it causes a detachment of the tissue and a hyperaemia of the treated area, with a consequent improvement of both the blood flow and the cellular metabolism. It provides a lipolytic action, through the mechanical action of the rollers that leads to a "squeezing" of the panniculus adiposus with subsequent emptying of the adipocytes and mobilization of fibrotic accumulations (which take the form of nodules) of cellulite.

Draining action: the intermittent vacuum pressure allows to give an important and targeted stimulus to the lymphatic system. ENDOSHAPER stimulates blood vessels with small contractions to promote lymphatic return, discharge toxins and eliminate interstitial fluid accumulations by pushing them towards the lymphatic ganglia.

Toning action: it stimulates fibroblasts to produce new fibres, thus improving skin texture, firmness, and elasticity. The Endoshaper device used for facial and décolleté treatments, with the vacuum applicators-cups of 3 different sizes, improves tissue oxygenation and stimulates the blood and lymphatic circulation, restoring a smooth and luminous appearance to asphyxiated and hypotonic skin.

TECHNICAL SPECIFICATIONS

EQUIPMENT NAME	Endoshaper
EQUIPMENT DESCRIPTION	Electronic device for endodermic massage
MODEL	Dec 30 UP
CATEGORY	Device for endodermic treatment
PROGRAMS	9 Localized preset programs PREPARATION PROGRAM POST-WORKOUT MUSCLE PROGRAM LOCALIZED REDUCING BODY FIRMING LOCALIZED FIRMING EDEMATOUS CELLULITE FIBROUS CELLULITE SCLEROTIC CELLULITE FACE Customizable programs Clients Database
TECHNICAL SPECIFICATIONS	Power supply: 230 V 50-60 Hz Fuses: 2x 6,3 AT Maximum consumption: 90 Watt Max suction: 800mBar Digital suction adjustment: MIN and MAX Operating modes: continuous and pulsed (0,1Hz - 5Hz) Language: Italian - English - French - German – Spanish Visualization of the real applied suction by sensor Overall Dimension: 63,5 x 34,5 x 30,5 cm Net weight: 8 Kg (without trolley)
GENERAL FEATURES	PROTECTION CLASS: I APPLIED PART: BF type
USER'S INTERFACE	Tablet
WORKING MODE	Contact by handpiece with rollers and vacuum cups
SAFE SYSTEMS	SAFETY AGAINST OVERVOLTAGE SAFETY AGAINST SHORT CIRCUIT SUCTION SYSTEM PROTECTION FILTER SAFETY AGAINST SUCTION FAILURE